

INSTRUCTIONS FOR USE




NewGen Surgical Sponge Bowl 32oz

Please read all information carefully.

Intended Use

The NewGen Surgical Sponge Bowl is a plant-based, single-use, sterile O.R. accessory designed to hold fluids and instruments during a surgical procedure.

www.newgensurgical.com/ifu

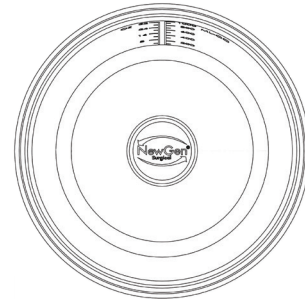

Warnings and Precautions

The NewGen Surgical O.R. bowls are manufactured with natural materials. With these materials, some discoloration, texture variation, or fiber inclusions may be present. Cosmetic appearance does not affect the performance nor sterility of the product.

To ensure performance, do not submerge in water or other liquids. Do not use excessive force.

This bowl is manufactured and sterilized for single-use only.

Do not reuse, reprocess, or resterilize. Reuse, reprocessing, or resterilization may compromise the structural integrity of the medical bowl and/or lead to medical bowl failure.

Store in a cool, dry place.

Do not use if product is damaged. Do not use if package is open or damaged.

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Instructions for Use

Follow recommended healthcare procedures.

	Sterilized using ethylene oxide
	See Instructions for Use
	Do not reuse
	Batch code
	Use by YYYY-MM-DD or YYYY-MM
	Manufacturer

	Catalog number
	Do not use if package is opened or damaged
	Latex-free
	DEHP-free
	PVC-free

REF NGS32SB

NewGen Surgical, Inc.
San Rafael, CA 94901 USA
855.295.4500
www.newgensurgical.com

© 2021 NewGen Surgical, Inc.
Manufactured in China

45-0402-A